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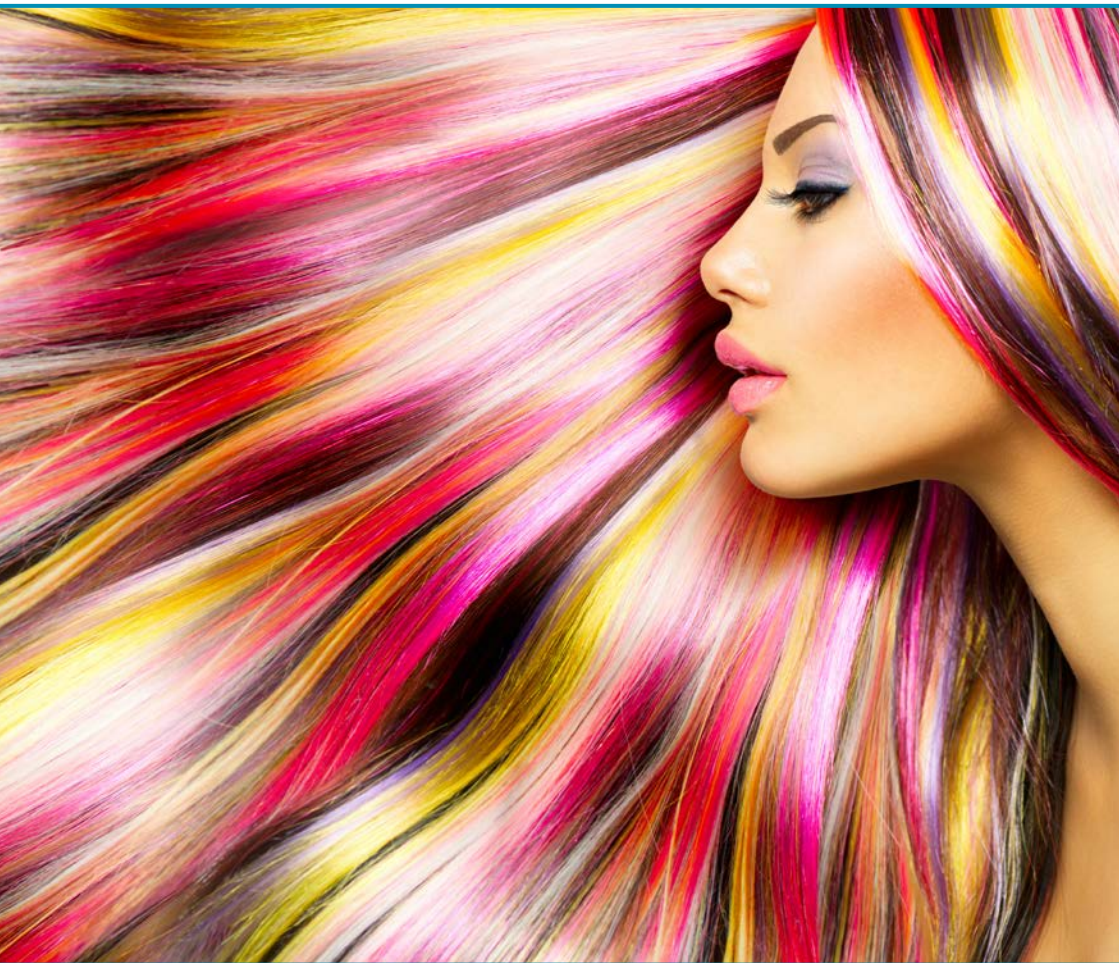
◆ Processing Instructions

Coloring Polyethylene

Our transfer papers can be used to color polyethylene materials like Subortholen and Eurolen Plus+, in order to give them an attractive appearance.

Technique:

- Heat up the polyethylene with a heating plate or an oven (at the temperature given in the manufacturer's instructions) until it turns completely transparent. Cut the transfer paper to shape and cover the PE plate with it, with the print on the paper facing the plate. Smooth out all wrinkles working from the center outwards.
- Close the lid of the heating plate, or close the oven (as applicable).

Coloring Straps

Our transfer papers can also be used to color straps, e.g. straps made of synthetic fibers.

Technique:

- Use a standard electric iron (ideally no steam iron). Set the iron to the highest temperature.
- Cut the transfer paper to shape and place it on the fabric with the printed side facing down, towards the fabric. Iron the paper onto the fabric with even pressure.

Coloring Padding Materials

Our transfer papers can also be used to color padding materials, e.g. Tepefom.

Technique:

- Use a standard electric iron (ideally no steam iron). Set the iron to medium heat, otherwise the Tepefom may get deformed.
- Cut the transfer paper to shape and place it on the fabric with the printed side facing down, towards the fabric. Iron the paper onto the fabric with even pressure.

Wilhelm Julius Teufel GmbH
Robert-Bosch-Straße 15
73117 Wangen
Deutschland/Germany
Phone: +49 (0)7161 15684-0
Fax: +49 (0)7161 15684-333
www.teufel-international.com