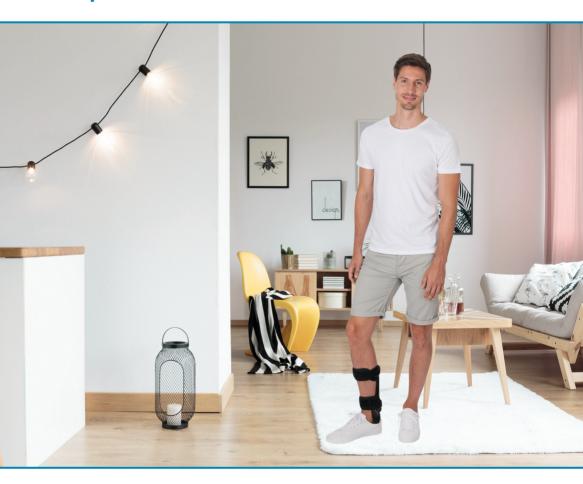
Return-to-Walk Step-On 100

Gebrauchsanweisung User Manual

◆ Inhalt

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Contents

Description	EN-2
Indication	EN-2
Contraindication	EN-2
Function	EN-2
Reimbursement Code	EN-2
Medical Product	EN-3
Safety	EN-3
REF/Sizes	EN-4
Scope of Delivery	EN-4
Accessories	EN-4
Applying the Orthosis	EN-5
Comfort/Hygiene	EN-6
Materials	EN-6
Product Care/ Cleaning/Maintenance	EN-6
Storage/Disposal	EN-7
Legend to Symbols	EN-7

Description

The Return-to-Walk Step-on 100 is an AFO with an individually adjustable joint. The dorsiflexion of the foot can be manually adjusted according to the neurological deficiencies of the patient. The thermoformable sole allows the orthosis to be customized to fit the geometry of the patient's shoe. Patients should only exhibit light spasticity. Wearing socks is recommended.

Indication

- Moderate drop foot
- Peroneal paralysis without spasticity
- Peroneal paralysis with spasticity
- Follow-up treatment of stroke
- Hemiplegia
- Poliomyelitis

Contraindication

- Distinct foot supination, caused by e.g. severe spasticity
- Foot deformities
- Moderate to severe edema formation in the lower extremities

Function

Prevention of foot drop during swing phase to minimize the risk of falling. In addition, the patient will regain a more physiological gait pattern and a quicker pace.

The joint can also be adjusted individually. This makes it possible to actively support the neurological regenerative process.

Reimbursement Code

DE HMV: 23.03.02.6026

Medical Product

The Return-to-Walk Step-On 100 is a medical device.

◆ **A** Safety

- Before this device is used, the orthosis
 has to be individually adapted to the
 patient by an experienced orthopedic
 technician and the patient has to be
 trained in the orthosis' use according to
 the doctor's instructions.
- Incorrect selection, adaptation, application, and/or failure to monitor the correct fit of this product may cause health damage.
- Check orthosis for damage (such as loose connections, fasteners, belts, etc.) each time it is put on.
- Do not remove the textile label from textile products.
- In case you suspect any performance malfunctions (e.g. an apparent reduction or lack of effect), immediately call on your doctor or orthopedic technician.
- All serious incidents that occur in connection with this product have to be reported to the Wilhelm Julius Teufel GmbH and the competent authority of the member state in which you are established. A "serious incident" is any incident that caused, may have caused, or may cause, directly or indirectly, one of the following consequences:

- a) death of a patient, user, or other persons
- b) temporary or permanent health deterioration of a patient, user or other persons
- c) a serious public health threat.
- Never wear the orthosis on broken skin.
- The orthosis must be checked for proper function, correct fit and potential damage by a doctor or orthopedic technician at regular intervals. The timing of these intervals has to be defined individually for each patient.
- Should the product have been subject to disproportionate stress, it must be checked for potential damage by an orthopedic technician before further use.
- Harmful pressure caused by the orthosis may cause skin injuries or disrupt blood flow and must be avoided.
- This product is intended for use on one person. It may not be reused on another person.

◆ REF/Sizes

Return-to-Walk Step-On 100, Medial

REF Left	REF Right	Size	Shoe Size	Weight Limit/kg
22.390.700.02	22.390.700.12	S-M	35-40	100
22.390.700.03	22.390.700.13	L-XL	41-46	100

Return-to-Walk Step-On 100, Lateral

REF Left	REF Right	Size	Shoe Size	Weight Limit/kg
22.390.800.02	22.390.800.12	S-M	35-40	100
22.390.800.03	22.390.800.13	L-XL	41-46	100

Scope of Delivery

REF links	Product
23.390.X00.XX	Return-to-Walk Step-on100 incl. Allen key
90.395.040.10	Return-to-Walk Step-on 100 UserManual

Accessories

REF	Produkt
22.390.700.97	RtW Step-on100 Replacement Part Kit
22.390.700.98	RtW Step-on100 Spare Pad&Strap Kit Lt
22.390.700.99	RtW Step-on100 Spare Pad&Strap Kit Rt

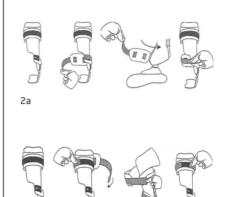
Applying the Orthosis

- **1.** Determine the shoe size of the patient and select the size of the orthosis accordingly.
- **2.** For the initial fitting the patient should be seated and have his knee bent at an angle of approximately 90° (fig. 1).

1

3. Once the lower leg and the foot have been positioned correctly in the orthosis, the length of both calf straps and the according tibia pads can be adjusted individually. To achieve a good fit the lower calf strap should be closed befor the upper calf strap. Be careful not to over-tighten the straps in order not to disturb blood circulation (fig. 2a/b).

4. For the final adjustment the patient should put on his shoe while wearing the orthosis. Use a hex key to adjust the joint resistance for dorsiflexion (fig. 3). Turning the screw clockwise will increase the joint resistance. Turning the screw anti-clockwise will make foot movement in dorsiflexion easier. To finish the adjustment of the orthosis you can customize the sole for the patient by thermoforming.

Comfort/Hygiene

Do not wear the orthosis directly on bare skin, but instead over thin clothing (e.g. a sock of appropriate length).

Materials

Orthosis, sole, calf frame:

Polypropylene

Padding, calf strap:

Polyamide, woven textile

Product Care/ Cleaning/Maintenance

⚠ Regularly perform a visual inspection of the orthosis and especially of the compression spring. To do so, unscrew the spring and check its condition. If necessary, re-grease the spring (machine grease) or replace it. (RtW Step-on100 Replacement Part Kit, REF 22.390.700.97). The spring in the replacement joint kit is already pre-greased.

For active use of the orthosis we recommend conducting an inspection every 6 months, for moderate use every 12 months.

• For the textile component:

Hand wash only.

Do not bleach.

Do not tumble dry.

Do not iron.

Do not dry-clean.

For the sole and the rigid calf frame:
 Clean with a wet cloth.

Storage/Disposal

Store the product in a dry place. Protect from dust, dirt and moisture. This product does not contain any environmentally hazardous substances. It can be disposed of with household waste.

Legend to Symbols

Manufacturer

Date of Manufacture

Distributor

Importer

EC Authorized Representative

Lot Number

Part Number

Serial Number

UDI Data Matrix

Follow User Manual

Caution

CE-Marking According to European Union Council Regulation 2017/745 Concerning Medical Devices (MDR)

Single Patient, for Multiple Use

Store in a Dry Place, Protect from Moisture

Protect from Sunlight

Temperature Limit

Humidity Limit

Atmospheric Pressure Limit, for Transport and Storage

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Wilhelm Julius Teufel GmbH Robert-Bosch-Straße 15 73117 Wangen Deutschland/Germany