WillowWood

**LimLogic®
Communicator**

Clever verbunden.

Software Installation

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⚠ Do not plug in the communicator until promoted.

Recommended Setup - Windows

Operating System	Windows® 7 or 8.1 with .NET Framework 4.5.1 installed
CPU	Intel® i3 or AMD® A8 or Phenom II X3 at 2.0 GHz or faster
Memory	2 GB RAM
Hard Drive	1 GB of free space
Graphics	DirectX 11 or OpenGL 4.0 compatible video card. Support for at least 1024 x 768 resolution and 1GB memory minimum. Use of vendor provided graphics drivers is highly recommended.
Other	USB port. In order to access updates and additional features, network and/or Internet connectivity is highly recommended. Use on a Mac or through Windows emulation is not supported.

Your LimbLogic Communicator software will perform well if installed on your existing OMEGA® System computer.

1. Download and run the Communicator software installer from <http://www.willowwoodco.com/products-and-services/elevated-vacuum/limblogic-communicator> under the **Resources & Related Products** section.
2. If the prompt shown at right appears, click **Yes** to continue the installation.

3. The **Welcome** screen will appear automatically. Click Next.


4. When the **License Agreement** screen appears, please review the agreement, then click **I Agree**.

5. On the **Choose Install Location** window, click on **Install**.

Changing the installation location from the default location that is shown here is not recommended.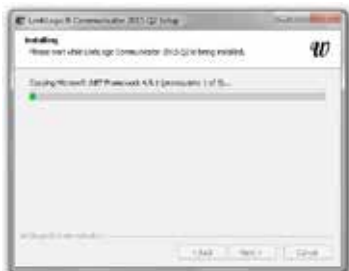

6. As the installation progresses, its status is displayed on the screen.

7. When prompted, insert the **Communicator** into a USB port on your PC. Click **OK**.

8. When the **Completing the LimbLogic Communicator Setup Wizard** window appears, click the **Finish** button.

9. A LimbLogic Communicator icon will appear on your computer's desktop.

Vertrieb durch:
Distributed by:

Wilhelm Julius Teufel GmbH
Robert-Bosch-Straße 15
73117 Wangen
Deutschland/Germany
Phone: +49 (0)7161 15684-0
Fax: +49 (0)7161 15684-333
www.teufel-international.com

WillowWood™

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
www.willowwoodco.com

Ohio Willow Wood Company B.V
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands